

NEW ACUVUE®
vita™
BRAND CONTACT LENSES



Lens Details	
Lens material	Senofilcon C
Diameter	14mm
Base Curve	8.4mm/8.8mm
Technology	HydraMax™
Dk/t ⁵	147 (for -3.00D)
Water content	41%
UV blocker†	Class 1
Center thickness at 3.00 Dmm	0.07 mm (for -3.00D)
Inside-out mark	Yes
Visibility tint	Yes
Powers	-0.50D to -6.00D (0.25D steps) -6.50D to -12.00D (0.50D steps) +0.50D to +6.00D (0.25D steps) +6.50D to +8.00D (0.50D steps)

For more information, visit: www.acuvue.com.my

*Important information for contact lens wearers: ACUVUE® Brand Contact Lenses are available by prescription only for vision correction. ACUVUE® VITA™ Brand Contact Lenses are available by prescription only for vision correction as a daily wear lens with one-month recommended replacement. An eye care professional will determine whether contact lenses are right for you. Although rare, serious eye problems can develop while wearing contact lenses. To help avoid these problems, follow the wear and replacement schedule and the lens care instructions provided by your eye doctor. Do not wear contact lenses if you have an eye infection, or experience eye discomfort, excessive tearing, vision changes, redness or other eye problems. If one of these conditions occurs, remove the lens and contact your eye doctor immediately. For more information on proper wear, care and safety, consult your eye care practitioner for more information. ACUVUE® VITA™ and HydraMax™ are trademarks of Johnson & Johnson Vision®, Johnson & Johnson Sdn. Bhd. 2019. MY303

f Acuvue Malaysia

Acuvue Malaysia

acuvue.com.my


ACUVUE® *vita*™ Monthly Contact Lenses Reinvented

FULL MONTH OF HYDRATION

Exceptional comfort even by the 4th week, with Hydramax™ Technology which helps to better maintain contact lens hydration vs. leading monthly lenses

HydraMax™ Technology

is a *NEW*, non-coated silicone hydrogel balanced to help


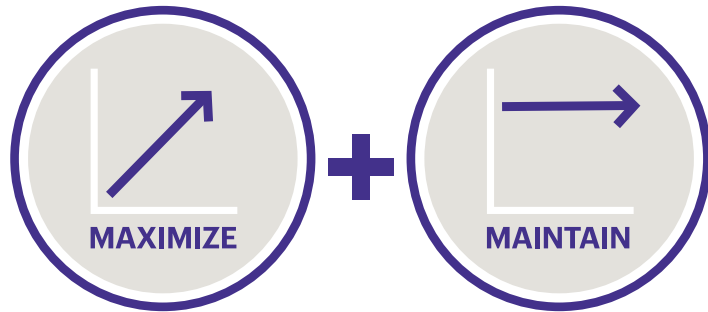


hydration throughout this contact lens - providing
reliable, superior comfort all month long

OPTIMISED EYE HEALTH WITH UV PROTECTION


The only monthly lens that meets
UV blocking standards among leading brands*

ACUVUE® VITA™ Class 1 UV blocks* approximately

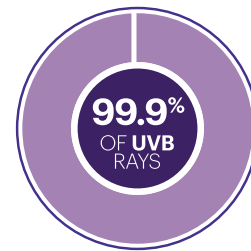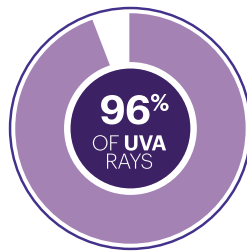

SMOOTH LENS EXPERIENCE

With Infinity Edge™ Design

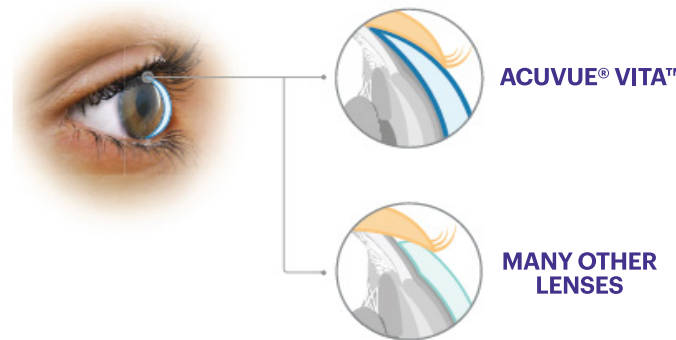


* JJVC Date on file All ACUVUE® Brand Contact Lenses have Class 1 or Class 2 UV-blocking to help provide protection against transmission of harmful UV radiation to the cornea and into the eye. UV-absorbing contact lenses are NOT substitutes for protective UV-absorbing eyewear such as UV-absorbing goggles or sunglasses because they do not completely cover the eye and surrounding area. UV transmission measured with -1.00 lens.

ACUVUE®



**A Full Month of
Hydration for Your Eyes!**
#GoSee

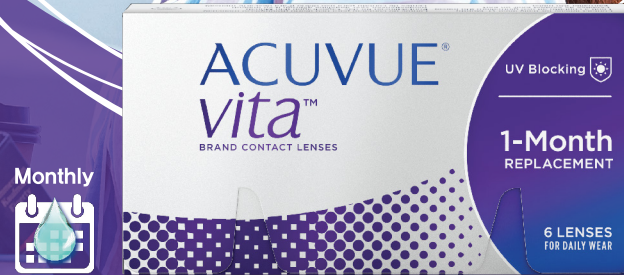

ACUVUE® VITA™ MONTHLY CONTACT LENSES

THE DO'S & DON'TS

OF CONTACT LENSES



DO

wash your hands and put your contact lenses in before starting your personal care routine.

DO

inspect your contact lenses before putting them in to be sure they are moist, clean and free of any nicks or tears.

DO

stay prepared by keeping an extra pair of contact lenses with you at all time.

DO

establish a replacement and reorder routine for your contact lenses.



DON'T

wear your contact lenses beyond the prescribed replacement schedule.

DON'T

sleep in your contact lenses unless your eye care professional says you can.

DON'T

share your contact lenses with anyone.

HOW TO PUT ON

ACUVUE® VITA™ MONTHLY CONTACT LENSES IN 4 EASY STEPS

1



WASH YOUR HANDS

Wash your hands with soap then rinse and dry your hands with a lint-free towel.


2



CHECK CONTACT LENSES ISN'T INSIDE OUT

Put the contact lenses on the tip of your finger and hold it up to the light. If the edges flare out, it's inside out.

3

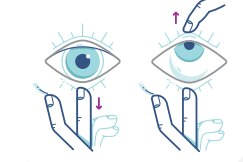

HOLD YOUR EYELIDS OPEN

Gently hold your upper eyelid with one finger and carefully pull down your lower eyelid with your other fingers.

4




PUT ON CONTACT LENSES IN YOUR EYE

Look upward as you place contact lenses on eye. Slowly release your eye lid and blink 2-3 to allow contact lenses to settle. Repeat for your other eye.

HOW TO REMOVE

ACUVUE® VITA™ MONTHLY CONTACT LENSES IN 4 EASY STEPS


1

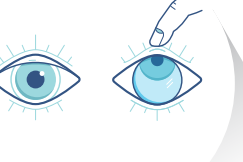

ENSURE HANDS ARE CLEAN & DRY

Look up and carefully hold eyelid with one finger.


2

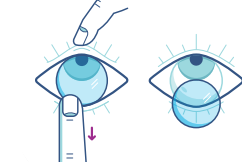

BRING INDEX FINGER TO YOUR EYE

Slowly until you touch the lower edge of the contact lens. Slide contact lens down to lower white part of your eye.

3

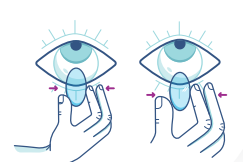

PINCH CONTACT LENS GENTLY

Between your thumb and index finger to remove it. Repeat for your other eye.

4



CLEAN & STORE LENSES

Make sure to clean and store your monthly contact lenses properly.

CLEANING & CARE

FOR YOUR MONTHLY CONTACT LENSES



WASH YOUR HANDS



Ensure your hands are washed, cleaned and dried to avoid irritation.

CLEAN CONTACT LENS AFTER REMOVAL



For monthly lenses, always clean your contact lenses immediately after you remove them.

BEGIN WITH THE SAME EYE



Establish a habit of starting with the same eye. This way you won't mix them up, since each eye has its own prescription.

NEVER USE TAP WATER TO CLEAN A CONTACT LENS OR CONTACT LENS CASE



Refer to your contact lens solution instruction guide for further information about how long to soak your contact lenses or how often you should change the solution if you don't wear them every day.