

Now meeting the needs of even more presbyopes!

1

UNIQUE  
PLATFORM

Renovation<sup>®</sup>  
MULTIFOCAL

Our original  
spherical base curve design.

2

PROVEN  
DESIGN  
OPTIONS

Renovation<sup>®</sup>E  
MULTIFOCAL


A low eccentricity base curve option.

### Flexibility meets CONTROL

Flexible, front-surface eccentricity control allows patients to translate into the near zone and corrects for aberration to eliminate the occurrence of optical distortion

Flexible distance/intermediate zone control accommodates small pupils to enhance near vision and large pupils to reduce flare and glare

Flexible automated thickness control reduces lens mass for maximum centration and wearing comfort even at higher add powers

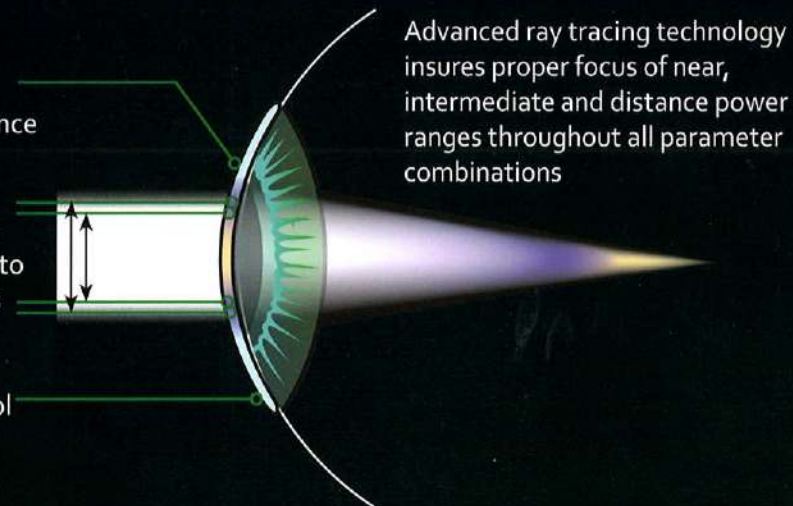

Advanced ray tracing technology insures proper focus of near, intermediate and distance power ranges throughout all parameter combinations

*Specially developed with the needs of the mature presbyope in mind, Renovation's flexible platform puts you in control of all your presbyopic patients.*

#### Available Parameters

**Power:** +/-20.00D in .25 steps

**Add:** +1.00 to +3.50D in .25 steps

**Diameter:** 9.0 to 10.0mm in .10 steps

**Base Curves:** 6.90 to 8.50mm in .05 steps

**Also available in toric designs!**

Distributed by

**OCULUS**

**Customer Service Hotline:** Singapore (65) 6284 5922  
Malaysia (603) 5635 9737 Hong Kong (852) 2503 5511  
China (8621) 6485 3164 Indonesia (6221) 634 8244

# Empirical Fitting in 4 Simple Steps!

EASY FITTING & REFERENCE GUIDE

# Renovation®

# Renovation®E



**DETERMINE THE BASE CURVE FIT BY REFERENCING THE FLAT K AND THE AMOUNT OF CORNEAL CYLINDER PRESENT**

**If Corneal Cylinder is:**     **Select Base Curve:**  
Spherical to 1.25D ..... On Flat K  
1.50 to 2.25D ..... 0.25D Steeper than K  
2.50 to 3.00D ..... 0.50D Steeper than K

**DETERMINE THE BASE CURVE FIT BY REFERENCING THE FLAT K AND THE AMOUNT OF CORNEAL CYLINDER PRESENT**

**If Corneal Cylinder is:**     **Select Base Curve:**  
Spherical to 1.25D ..... 0.50D Steeper than K  
1.50 to 2.25D ..... 0.75D Steeper than K  
2.50 to 3.00D ..... 1.00D Steeper than K

**DETERMINE THE DIAMETER BY THE BASE CURVE SELECTED**

**If Base Curve is:**     **Diameter Selection:**  
8.50 to 8.45mm ..... 10.0mm  
8.40 to 8.20mm ..... 9.6mm  
8.15 to 7.50mm ..... 9.5mm  
7.45 to 7.20mm ..... 9.2mm  
7.15 to 6.90mm ..... 9.0mm

**DETERMINE THE DIAMETER BY THE BASE CURVE SELECTED**

**If Base Curve is:**     **Diameter Selection:**  
8.50 to 8.45mm ..... 10.0mm  
8.40 to 8.20mm ..... 9.6mm  
8.15 to 7.50mm ..... 9.5mm  
7.45 to 7.20mm ..... 9.2mm  
7.15 to 6.90mm ..... 9.0mm

**DETERMINE THE DISTANCE POWER**

Compensate for any vertex change (sphere powers of (+/-) 4.00 diopters or greater) and adjust for any tear layer change generated from going steeper than flat K.

**DETERMINE THE DISTANCE POWER**

Compensate for any vertex change (sphere powers of (+/-) 4.00 diopters or greater) and adjust for any tear layer change generated from going steeper than flat K.

**DETERMINE THE NEAR ADD POWER**

Add 0.25D to the spectacle add power. If spectacle add power is +2.00, then contact lens near add power will be +2.25.

**DETERMINE THE NEAR ADD POWER**

Add 0.25D to the spectacle add power. If spectacle add power is +2.00, then contact lens near add power will be +2.25.

# OCULUS

For more information, go to [www.ocusgpc.com](http://www.ocusgpc.com)  
or write to [professional\\_service@oculuslens.com](mailto:professional_service@oculuslens.com)